

OVERLINE

ITALIA



élite Resolution Total Body **Pro**

perfect synergies for a Fabulous body



3238

élite Resolution Total Body **Pro**

SUPER Laser Remodeling Action **Cellulite*** Adiposity & Tone loss



An Avant-garde device based on a mix of technologies effective in the silhouette slimming, which thanks to the special handpieces can be also used in a synergic way or just singularly.

Allows a reducing, firming, toning, modeling and draining action, thanks to the lipolytic stimulating action of the Low Level Laser (LLLT) that helps directly, focused and non-invasive on the cell membrane, favoring the exchange and release of fat from the adipose cell.

The device offers also Endomassage to drain, Ultrasound 1MHz for a slimming action, Electrolipolysis to tone up, Radio Frequency for a regenerative activity, Fractional RF Stretch Marks to work on this imperfection and generate elasticity, Electroporation for an intensive action, Chrome Frequency for a rebalancing effect. We don't have to forget that being a complete device, there are also special handpieces for the face, to offer a total look response to any client.

Advanced**Plus**

- Revolutionary Synergy
- Laser Remodeling
- Multifunctional handpieces
- Ergonomic design
- Specific treatment for stretch marks
- Silhouette redefinition
- Specific Radio Frequency Mix
- Conveying active ingredients
- Integrated facial
- Toning action
- Relax option
- Intensive function

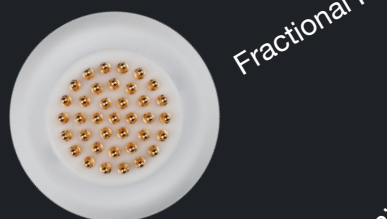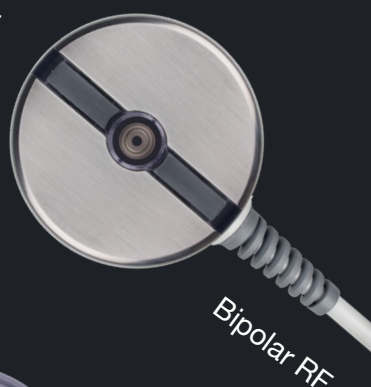

revolutionary Handpieces


ref. 3238

Dimensions W 21.66in x D 18.90in x H 45.28in
W 550mm x D 480mm x H 1150mm



synergy**Endo**

- Radio Frequency
Regenerating Action
- Endodermal massage
Draining Action
- Chrome Frequency
Heating / Rebalancing Action
- Low Level Laser Treatments
LLLT Stimulating Action

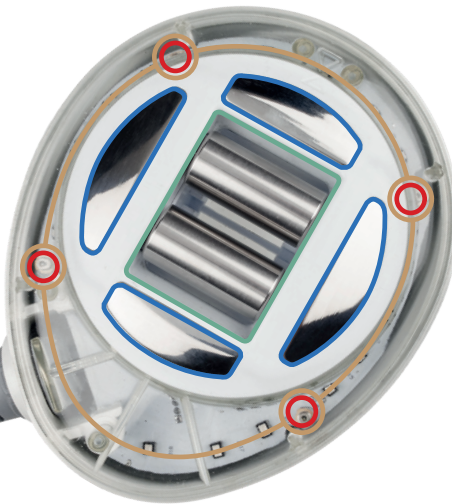

synergy**Ultra**


- Ultrasound
Slimming Action
- Electrolipolysis
Tonifying Action
- Electroporation
Intensive Action
- Chrome Frequency
Heating / Rebalancing Action
- Low Level Laser Treatments
LLLT Stimulating Action



Revolutionary Handpieces

synergyEndo

Revolutionary synergy that fights the imperfections of cellulite*, adiposity and sagging skin.

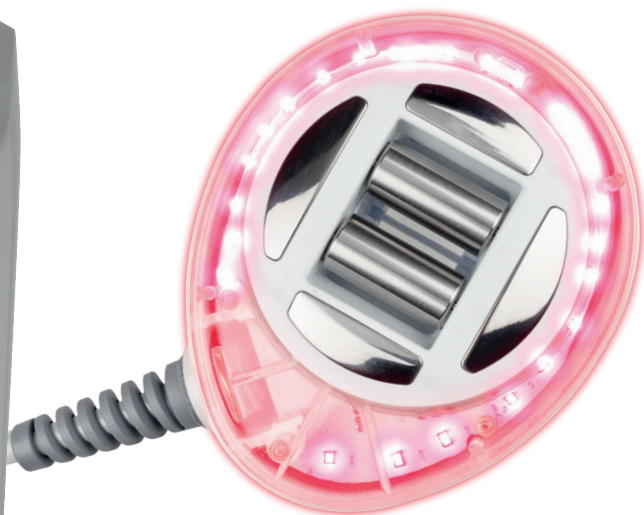

- Radio Frequency
Regenerating Action
- Endodermal massage
Draining Action
- Chrome Frequency
Heating / Rebalancing Action
- Low Level Laser Treatments
LLLT Stimulating Action

Thanks to the revolutionary handpiece, body treatments will always leave you speechless, its own ergonomic design allows every kind of beauty operator to held it very comfortably without getting tired, but concentrating on the body path and the goals. Synergy means result and in this case, thanks to the combination of different energies, getting progressive results will not be a problem. Imperfections due to Cellulite, Adiposity and Tone loss will look better thanks to the action of Multipolar Radio Frequency combined with Low Level Laser, that will prepare the skin and stimulate fibers to actively react. With the help of color waves of the Chrome Frequency and Endodermal Massage, the silhouette will never be the same.

* imperfections due to cellulite



RADIO FREQUENCY

Regenerating Action

Radio Frequency is a non-invasive technique that takes advantage of radio waves to determine a tissue remodeling with a particularly effective action to counteract the body and face tone loss.

ENDODERMAL MASSAGE

Draining Action

It's a particular massage technique that stimulating the arterial circulation and the connective net, brings benefits from the first treatment. There are two fundamental ways to act: with the decontracting action and the draining function.

CHROME FREQUENCY


Heating / Rebalancing Action

Chrome Frequency considers the imperfection as a discordant vibration of the skin cells that can be rebalanced taking advantage of energy and then the vibration of one or more specific colors.

LOW LEVEL LASER TREATMENTS

LLLT Stimulating Action

The LLLT treatment uses a ray similar to a single electromagnetic frequency wave pencil and predefined wavelength, to promote tissue stimulation and an effective action on body remodeling.

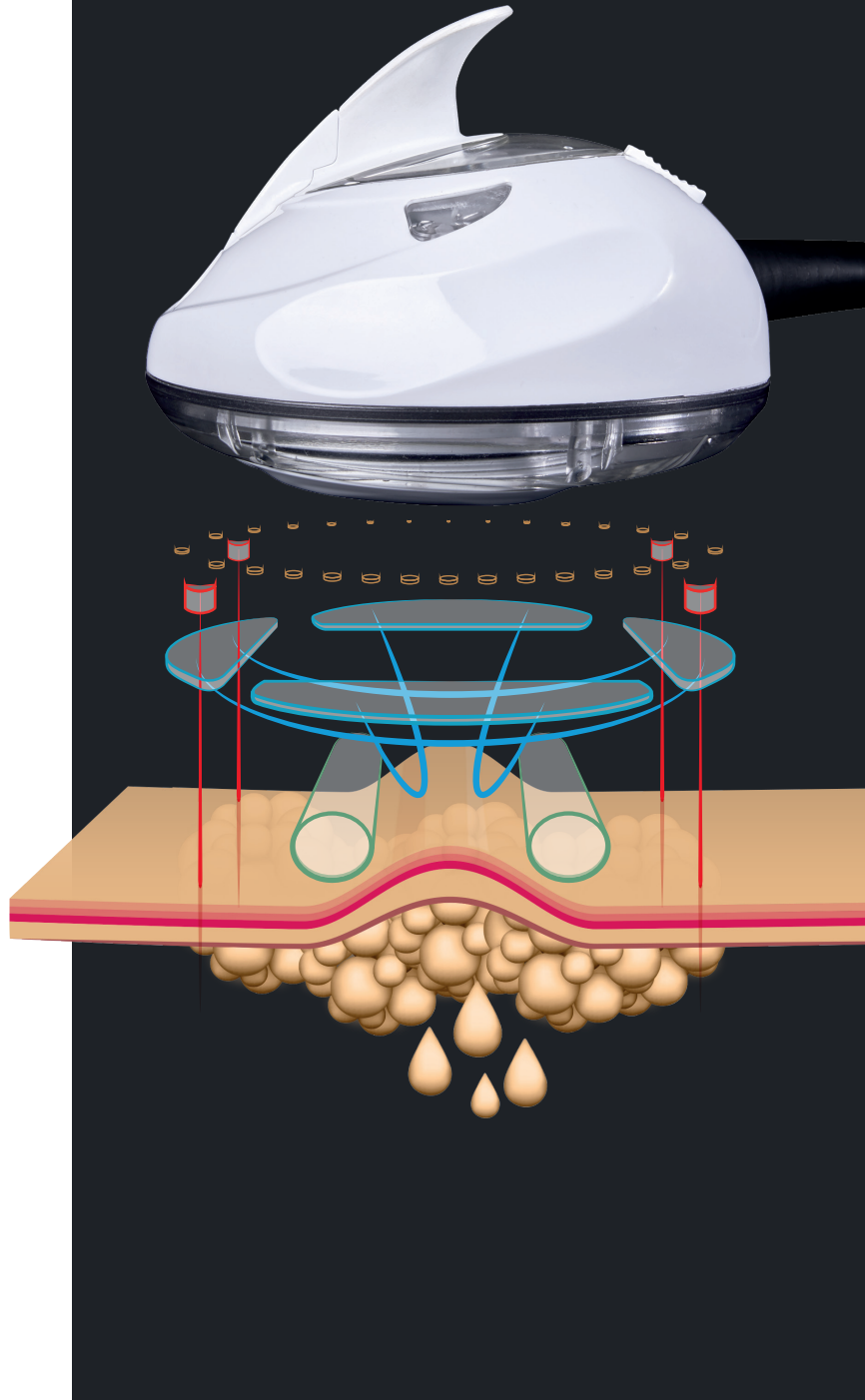

<p>RADIAZIONE LASER INVISIBILE EVITARE L'ESPOSIZIONE DIRETTA DEGLI OCCHI INVISIBLE LASER RADIATION AVOID DIRECT EYE EXPOSURE MAX10 mW/cm² - 980 nm EN 60825-1:2014-08</p>		<p>CAUTION LASER 3R</p>	
---	---	--	---

Revolutionary Handpieces

synergyUltra

Revolutionary synergy that fights the imperfections of cellulite *, adiposity and sagging skin.



- **Ultrasound 1 MHz**
Slimming Action
- **Electrolipolysis**
Tonifying Action
- **Electroporation**
Intensive Action
- **Chrome Frequency**
Heating / Rebalancing Action
- **Low Level Laser Treatments**
LLLT Stimulating Action

This Revolutionary handpiece is called Ultra because, thanks to the energy of specific actions, it is able to improve the most common imperfections that involve the silhouette of both female and male body. The combined action of Ultrasound and Electrolipolysis, it will be decisive in the remodeling, in particular of that area with too much tone loss. The heat of the Chrome Frequency and the LLLT will stimulate the fiber to revitalize. Electroporation is the best tool to help us to let absorbed the precious active principles in that area where we need the most. Tone loss and imperfections will be a far nightmare and your body will never be the same.

* imperfections due to cellulite



● Ultrasound 1 MHz

Slimming Action

The Ultrasound treatment is based on the periodic vibration of particles of a material way. Vibrations that propagate in the middle, with a finite speed. Ideal in the treatment against imperfections of adipocytes and against the imperfection caused by cellulite.

● ELECTROLIPOLYSIS

Tonifying Action

Electrolipolysis aim is to reduce the imperfection of cellulite and adipocyte deposits. The adipocytes are reduced in specific areas, without effort or discomfort, using low voltage electrical currents through skin plates. It is a non-invasive treatment. Electrolipolysis takes advantage of application of micro electric currents to treat effectively imperfection of cellulite.

● ELECTROPORATION

Intensive Action

Electroporation is a treatment that improves the penetration and it enhances the vehiculation of the dermal cosmetics in the deeper layer of the dermis, allowing to act in a direct and immediate way toward the skin regeneration process.

● CHROME FREQUENCY


Heating / Rebalancing Action

Chrome Frequency considers the imperfection as a discordant vibration of the skin cells that can be rebalanced taking advantage of energy and then the vibration of one or more specific colors.

● LOW LEVEL LASER TREATMENTS

LLLT Stimulating Action

The LLLT treatment uses a ray similar to a single frequency electromagnetic wave pencil and predefined wavelength, to promote tissue stimulation and an effective action on body remodeling.

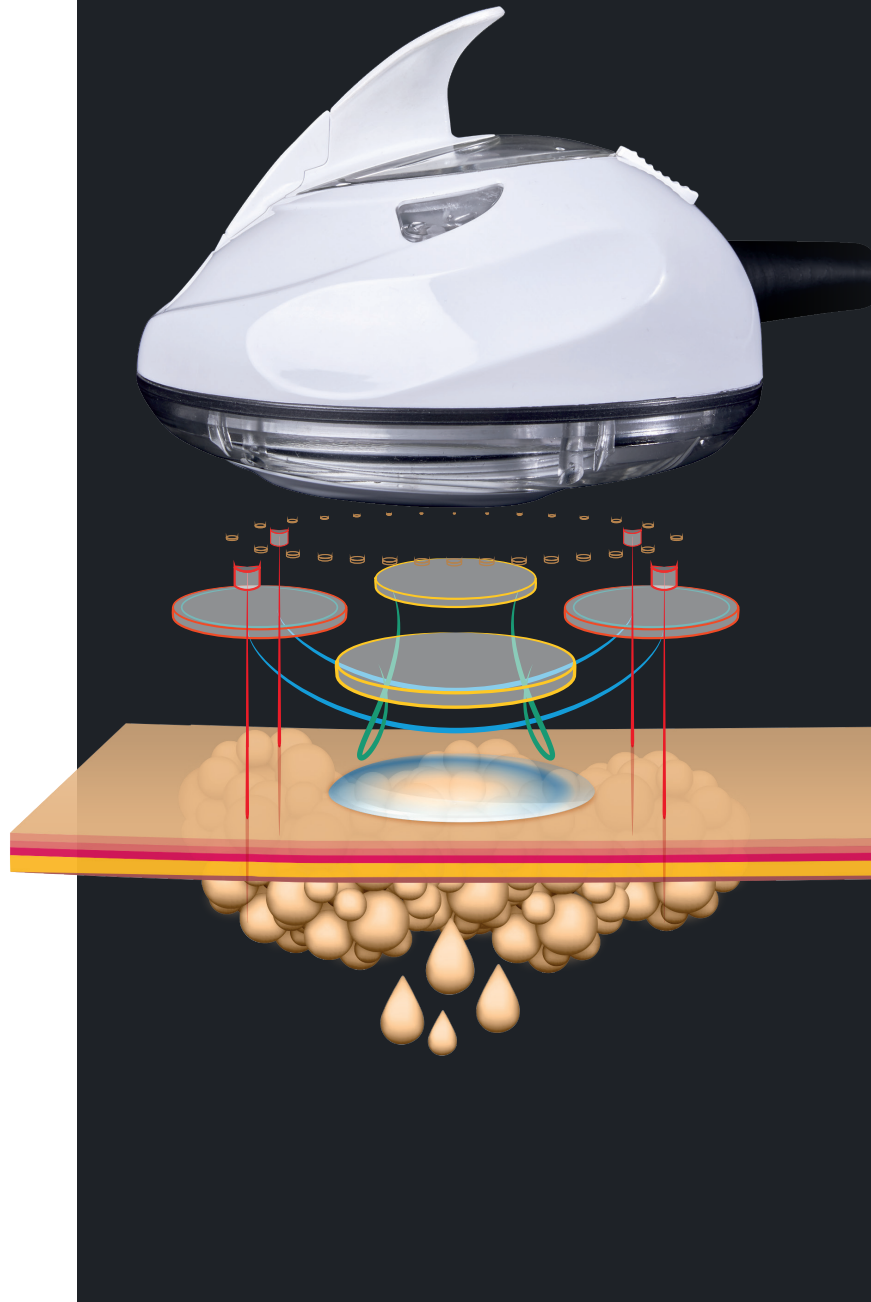

radio**Frequency**

to model & define the Body

the **Body**



MULTIPOLAR RF

Remodeling Action

The Multipolar RF takes advantage of both the methods Monopolar and Bipolar to bring the energy under the skin, moreover has the advantage to focus the beam of action in one point through the system of the four poles. In this way we can obtain higher temperatures both on the surface and in the deeper layers. The multipolar handpiece allows also to manage low power with a results of treatment time relatively longer without pain or discomfort for the client.



BIPOLAR RF

Firming Action

Biophysical Effect of Radio Frequency is based on the transformation of energy into heat. Increases the local metabolism, fastens the blood flow into the micro circle, eliminates the stagnation of liquids into the capillaries, frees the interstitial tissue from the edematous engorgement, re-establishes the temperature necessary to revive the lipolysis. Interrupts the fibrotic reaction in the sub-cutaneous layer, reduces the tone loss and the effect of cellulite imperfection.



FRACTIONAL RF

12pin STRETCH MARKS

The specific applicator creates a specific action, concentrating the strong action directly on the stretch marks, to stimulate the regeneration of the damage tissue fibers at the level of dermis layer.





RFSMART

1. The new handpiece RFSMART, designed by Overline research Lab, helps the beautician to perform Radio Frequency treatment.
2. Its main feature is to detect the temperature in the treated area, showing it on the display located on the handpiece, thus ensuring the operator is working in compliance with the regulations regarding the temperatures achieved during Radio Frequency treatments, issued by the Ministry of Health of Italy with decree-law n. 206 entered in force in January 2016.
3. The led integrated handpiece, allows synergistic performance of Chrome Frequency treatment which consists in the irradiation of the area of the body with the vibration of different colored lights, to obtain an aesthetic advantage.
4. Thanks to the advanced software implemented in the technology, the handpiece RFSMART recognizes the selected treatment and shows immediately the right handpiece to use, making immediate the visual of the handpiece to be used.
5. The display indicating the countdown of the time shows to the operator the correct time of execution on the treated area.



1. Smart and ergonomic handpiece for Radio Frequency treatments
2. Temperature detector on the treated area
3. Integrated LED for the Chrome Frequency
4. Indicator of execution time
5. Light indicator to show which handpiece has to be used

radioFrequency for **Face**

Total Body Pro includes handpieces born to be used exclusively on face



FRACTIONAL RF Deep Action

the FRF allows the homogeneous increase of the temperature at the level of dermis, causing a thermal action toward the collagen fibers and allows to rebalance both the surface and the deep fibers, reorganizing the collagen fibers in a physiological way. This action leads to the formation of new collagen thanks to a volumetric increase of heat.


Fractional RF is a Radio Frequency system able to match from resistive thermo-stimulation with plastic remodeling of the dermis layer up to resolving the texture irregularity, determining a rejuvenation of the skin to all its thickness.

FRACTIONAL RF 37 pin DEEP WRINKLES


The round shape handpiece studied in the Overline Laboratories, allows to concentrate the action of Fractional Radio Frequency achieving great results on the wrinkles immediately from the first treatment.

BIPOLAR RF Regenerating Action

This treatment with an extraordinary lifting effect, fights against the tone loss. Increasing the temperature of the connective layer using heat, fibroblast start to produce new Collagen and Elastin. Thanks to this effect, the wrinkles and the time signs will be smooth and the face will look naturally younger. The effects will be more consolidate after a couple of months of treatment.

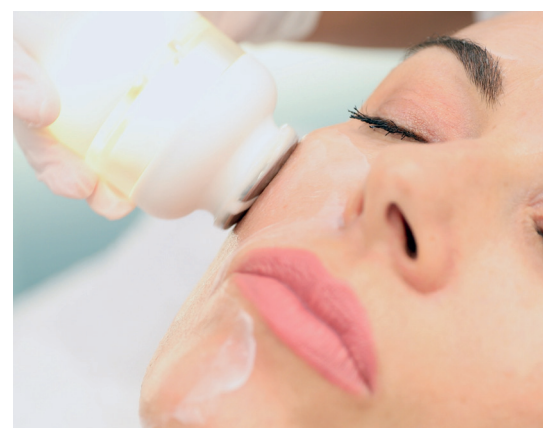


Models

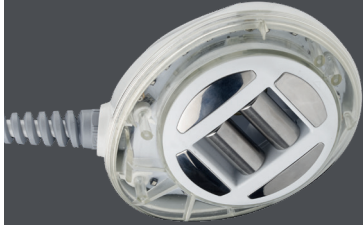


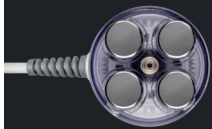
 Synergy
ENDO



 Synergy
ULTRA

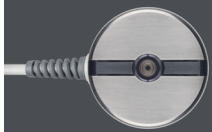


 Bipolar Body Radio
Frequency




 Multipolar Body
Radio Frequency



 Fractional Radio
Frequency 37 pin




 Fractional Radio
Frequency 12 pin




 Bipolar Face Radio
Frequency



 Multipolar Face Radio
Frequency



 Endodermal Face

 OPTIONAL



3238



3235



3239



3237



3247



Overline reserves the right to make changes to the equipment described, at any time and without prior notice. Overline equipment is designed and programmed to obtain results on imperfections without passing the allowed aesthetic limit.



iSol. Beauty Integrated cosmetics

Overline, provides concrete answers for all beauty needs with technological gems designed to act in a targeted manner on all issues and the integrated iSol. Beauty cosmetics. All designed to give the best results and an excellent beauty experience the customer of the beauty center.



Enhance the benefits of **Treatments**

Cleansing



CLEANSING SOFT MOUSSE

Dermocompatible Face & Body cleanser
ISO.FOAM.150 - 150ml / 5,1fl.oz
(Cabin & Home Care)



JOJOBA FACE & BODY SCRUB

Jojoba Microgranules Gel Scrub
ISO.SCRUB.200 - 250ml / 8,45fl.oz
(Cabin)

Body treatments



RF BODY EMULSION

Body Emulsion for RF
ISO.RF.500 - 500ml / 16,9fl.oz
(Cabin)



ULTRASOUND BODY GEL

Innovative Gel with Phosphatidylcholine
ISO.GEL.110 - 500ml / 16,9fl.oz
(Cabin)



OILGEL SYNERGY

RF & Massage
ISO.RFOIL.100 - 500ml / 16,9fl.oz
(Cabin)



RESOLUTION BODY CELL

Cellulite Liquid Crystals serum for Body Electroporation
ISO.POR.500 - 250ml / 8,45fl.oz
(Cabin)



RESOLUTION BODY TONE

Toning Liquid Crystal serum for Body Electroporation
ISO.POR.550 - 250ml / 8,45fl.oz
(Cabin)



HYDRABODY CELL

Cellulite Body Emulsion
ISO.BODY.100 Airless 200ml / 6,76fl.oz

HYDRABODY FIT

Toning Body Emulsion
ISO.BODY.200 Airless 200ml / 6,76fl.oz

(Home Care)

Face treatments



RFF EXTREME

Anti-Age Emulsion before Fractional RF
ISO.RFF.110 - Airless 150ml / 5,1fl.oz
(Cabin)



RF & STIM FACE EMULSION

Emulsion for RF and Stimol
ISO.RF.100 - 250ml / 8,45fl.oz
(Cabin)



RF FRAZIONATA ACTIVATOR

Anti-Age Serum after Fractional RF
ISO.RFF.120R - 10 Vials 3 ml / 0,1oz.
(Cabin & Home Care)



always at
the service
of beautician

OVERLINE

Avant-garde and Innovation are the ideal terminologies to identify the Overline Company. A historic Milan reality, undisputed leader in the production and marketing of hi-tech equipment for professional aesthetics.

It was born in 1989 and since then it can be simply described "A success story" that expresses one's style business through an ethical commitment and a commercial management attentive and responsible.



OVERLINE
ITALIA

A 100% Made in Italy production, which offers competitive advantages in terms of performance, safety and service to the customer. Overline technologies are distributed in the best beauty centers and beauty institutes throughout the Italian territory and in all European countries as well as in South America, Mexico, Africa and Asia



The terminologies adopted in the company information materials, to define technologies and characteristics, are those allowed on an international level and commonly used.

RESEARCH & DEVELOPMENT

Heart of the company, the Innovative Laboratory Research & Development where high performance technologies and simple use, provide effective answers to the needs of professional aesthetics operators. They are different systems, safer and multiple features for both face and body but also for epilation.



reliability
research
safety



CERTIFICATIONS



Overline is a Technology S.r.l. Brand, a company that engages itself constantly in the research and development of technologies, to offer to the professional beauty market, innovative systems that give visible results but at the same time respect the maximum safety of the operator and the customer. Today, Technology, thanks to its great experience, has obtained important certifications attesting the fullness compliance with international regulations and standards in production of biomedical equipment, electro-aesthetics and cosmetics. The certifications that make all the technologies at the highest level are the following:

Tüv Rheinland ISO 9001:2015. Represents the certification of the Quality Management System, recognized at international level, ensures that the equipment meets the management system required by law for the design, the production, the sales, the development and the research.

Tüv Rheinland EN ISO 13485:2016. The certification related to the Quality Management System for Medical Devices, it is a standard of Quality reference in the regulatory procedures that govern design, production and the placing on the market of medical devices.

Tüv Rheinland OHSAS 18001:2007. Relative to the standard for the management of health and safety of workers.



TAKE CARE OF YOUR WORK DON'T WORRY ABOUT THE REST, OVERLINE IS THERE FOR YOU



Overline devices are guaranteed by defects in material and workmanship, ex-works, for a period of 36 months according to the General Conditions of Sale*

We are associated with:



OVERLINE

ITALIA



mesa.comunicazione

Overline reserves the right to make changes to the equipment described, at any time and without notice. Overline equipment is designed and programmed to obtain results on imperfections without passing the allowed aesthetic limit.



For further information please contact us at number **+371 2 5132866** or visit our website **www.beautyequipment.lv** write to **beautyequipment.lv@gmail.com** and follow us on our Social Page

