OVERLINE
élite

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Programmi Benessere *Wellness programs*

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élite Resolution Face takes care of face, neck and décolleté blemishes. It's a device that encompasses the most advanced technologies for cleansing, oxygenating, toning and regenerating treatments. This device permits two distinct working methods to solve the detected imperfection:

- **Guided treatment:** *the beautician identifies the imperfection and so the device accompanies her at all stages, from the starting analysis through every step of the treatment, indicating the most suitable technology.*
- **Manual treatment:** *the beautician herself selects the type of the imperfection and chooses personally the treatment. If necessary, she can always count on the device, that displays the characteristics of each imperfection and proposes the solutions.*

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This treatment is pleasant and effective like a manual massage, but it also permits to treat the face more equally and it is less tiring for the beautician.

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The Chromo Frequency considers the skin imperfections as a disharmonious vibration of cells, which can be leaded to their correct balance using the energy of the colors. The Mask Light Face and the specific alginate masks can bring back the skin to its correct balance. The Chromo Frequency increases the final results of the other aesthetic treatments.

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The monitor shows the image of a face with the points where to put the handpiece for the value detection of skin hydration.

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One of the most important cleansing treatment is the "Brush", a gentle mechanical action performed by different types of brushes.

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Treatments: surface and deep cleansing - anti-spots - conveyance of active ingredients. 14 programs

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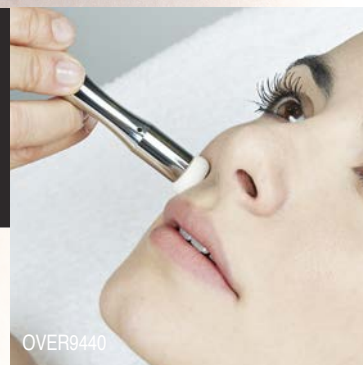
It is used to obtain a skin descaling action and also a iontophoresis action. Two programs: Descaling, Iontophoresis.

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This treatment allows the exfoliation of the skin layers, encouraging the cell renewal of the treated tissue. It increases also the skin permeability, supporting the absorption of the used cosmetics.

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The diathermic wave allows to restore a proper vascularization system in all the treated areas. The Diathermy also stimulates some natural processes, such as the production of collagen, elastin and hyaluronic acid, increasing their action.

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Through the emission of electrical impulses, the Electroporation contributes to open channels in the dermis to convey in depth any substance applied, chosen according to the imperfections to be treated.

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It allows the (electro-lifting) stimulation of the cutaneous microcirculation, by promoting nutrition, oxygenation and regeneration of the tissues.

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The bipolar Electrostimulation tones up the unstressed muscles gently and selectively.

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Cosmetics keeping up
with technology

ISO.FOAM.150
150ml
**CLEANSING
SOFT
MOUSSE**
Detergente
Dermocompatibile
viso&corpo

ISO.SCRUB.200
250ml
**JOJOBA
FACE &
BODY SCRUB**
Scrub in gel ai
microgranuli di
Jojoba

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