

OVERLINE

èlite Resolution **HD** Laser 808nm

2018



REVOLUTIONARY



new







LASER



THE REVOLUTIONARY **LASER** THAT TRANSFORMS THE WAY TO DO AESTHETICS

- Easy to use
- Ergonomic Handpiece
- Control System
- Operation identification
- Laser Security
- Operator Safety
- Executive Protocol
- Education


NEWS 2018


-  High Resolution Laser Matrix
-  Large Focus Sapphire 22x13mm
-  Freezing Effect
-  Video Educational

see page 11



9931-12L • 1200W
Large Focus Sapphire 22x13mm

 9931-12 • 1200W
Regular Focus Sapphire
13x13mm

 9931-10 • 1000W
Regular Focus Sapphire
13x13mm



VIDEO EDUCATIONAL

At the forefront in terms of service, Overline is always flanked by the professional aesthetic operator to guarantee a result of excellence.

Certain of our strength, we have thought about the usefulness to insert some educational videos in our equipment, designed to allow the operator to consult at any time the correct operating protocols.



TECHNICAL CHARACTERISTICS

- Equipment size in mm L550 x D480 x H1150 (L21.65in x D18.89in x H45.27in)
- Source: HRLM laser
- Wavelength 808nm
- Dimension of the Sapphire window: mm 22x13 (0.86in x 0.51in) Large / 13x13 (0.51in x 0.51in) Regular
- Emission: from 1 to 12 multi-spot pulses per second
- Emission: 1 to 3 single-point pulses fraction
- Pulse duration defined by the sophisticated system of choice of the working parameters within the limits of the current legislation
- Maximum energy density 40J/cm² max 300ms
- Cooling Handpiece from 5 ° to 15°C (41°F to 59°F)
- 36-months warranty (see page 13)

QUICK MAINTENANCE

Thanks to the advanced research developed by Overline, we have created a technology that requires minimal maintenance only every 36 months, it will be necessary to replace water and filters with simple operations illustrated below.

LASER EDUCATION

Laser technology, in compliance with current legislation, must be accompanied by a training course for the professional who puts himself in a position to know all about Laser technology, every little secret will be clarified, so you can transfer this training within your Center for any colleagues. Overline always attentive to this type of regulatory instructions, has long been providing its customers various types of training and preparation at any level in order to ensure the tools to perform their work safely. Our responsibility is to put all Professional operators laser, in condition to work in the most possible correct way by respecting 100% your customer that is requesting such type of service.



LASER HANDPIECE

COMPACT • ERGONOMIC • LIGHT
RELIABLE • FUNCTIONAL • MULTI ZONE
SAFE • PROFESSIONAL • EASY TO HANDLE

Anti-fall strap, laser emission indicator lights and cooling, practical angular button for different way to use.












Laser Collection 2018

Overline, thanks to its Research and Development laboratories, has succeeded in creating a wide range of laser hair removal equipment. Each model differs from the others according to the technical specifications and consequently to the performances. Elite Resolution HD LASER 2190-12L is the most complete and technically advanced system compared to other technologies. Thanks to the complete variety of equipment, Overline is able to meet any need to offer to the professional operator of the Beauty Center a product always cutting edge with extraordinary results.



2190-12L

1200W

-  High Resolution Laser Matrix
-  Large Focus Sapphire 22x13mm
-  Freezing Effect
-  Video Educational
-  Macro-cam
-  12 Spot per second
-  Memory Card
-  Technical Evaluation
-  Price List












9931-12L



2190-12

1200W

-  High Resolution Laser Matrix
-  Regular Focus Sapphire 13x13mm
-  Freezing Effect
-  Video Educational
-  Macro-cam
-  12 Spots per second
-  Memory Card
-  Technical Evaluation
-  Price List












9931-12



2190-10

1000W

-  High Resolution Laser Matrix
-  Regular Focus Sapphire 13x13mm
-  Freezing Effect
-  Video Educational
-  Macro-cam
-  12 Spots per second
-  Memory Card
-  Technical Evaluation
-  Price List



9931-10



2190-08

800W

-  High Resolution Laser Matrix
-  Regular Focus Sapphire 13x13mm
-  Comfort o Freezing Effect
-  6 Spots per second
-  Technical Evaluation



9931-08



High Resolution Laser Matrix

All new generation Laser equipment use a wavelength of 808nm corresponding to one laser beam of pure red light. Especially effective on the pigments of the hair bulb that absorb the Light up to about 98%. Thanks to the increased 1200W power we will have further precision in hitting the hair.



Large & Regular Sapphire Focus

REGULAR: thanks to its 1200W of power, it allows to get a decisive action on the area to be treated. In addition, the greater reduction in pulse width combined with the Freezing Effect cooling system allows more comfortable treatments. LARGE: innovative design with doubled spot, allows reducing treatment times, especially on large areas, maintaining the same effectiveness, but requiring fewer steps.



Freezing Effect

It simply means that we have introduced an advanced cooling system that makes the experience of the Overline Laser epilation, a pleasant sensation of freshness and protection for the skin. Also cooling it will no longer influence the effectiveness of the handpiece but at the contrary will emphasize the result of the progressive epilation treatment.



Video Educational

At the cutting edge in terms of service, Overline has always worked alongside the professional aesthetic operator to guarantee a result of excellence. Certain of our strength, we have thought about the usefulness of inserting some educational videos within our equipment, designed to allow the operator to consult the correct operating protocols at any time.



12 spots per second

With a view to optimization and improvement of the performance, the execution of the Overline Laser treatment, provides the use of a multispot methodology that allows up to 12 spots in a fraction of a second. All this puts the aesthetic professional operator in conditions to be able to work safely and improve time and effectiveness of the treatment and its final result.



Macro-Cam

The cam integrated in the Laser technology is a real 200X magnification microscope of the desired area to be analyzed. Through this tool, we create a strong visual impact on the client and a real perception of his imperfection. The image is also transmitted on our 10 "display, for a better view.



Memory Card

The memory card is nothing but a practical and effective "Personal Card" that allows you to store any type of information related to the type of treatment we are carrying out and possibly customize it or run it instantly. To be able to have access to data within the card, all you have to do is insert it into the slot.



Technical Evaluation

With our 10" touch screen we can customize any treatment thanks to a detailed evaluation. The software offers the possibility for the professional operator to enter all the personal data of our customer, related to the epilation service. We are able to recognize four important families: the phototype according to Fitzpatrick, the color of the hair, the size of the hair and finally the piliferous density.



1

DETERSION

iSol. Beauty Cleansing Mousse helps to prepare in a optimal manner the area to be treated, cleaning the tissue perfectly to get the ideal result

COVER

It is very important to remember to isolate with a white pencil all the tattoo and mole, included the lips contour, to avoid Laser to recognize the area, creating any unpleasant situation to the client

2



3

COVER

To guarantee the best protection and cote to the area covered with the white pencil, we intervene also with a white medical tape or special round white tape available in any shop



S T E P B

MACRO CONSULTATION

A very careful and detailed evaluation helps to facilitate the path of the treatment, the supplied macrocamera allows to zoom in up to 200X

4



5

TECHNO EVALUATION

Thanks to the 10" touchscreen display, the operator is able to define the main parameters (skin phototype, color, dimension and density of the hair) to personalize the treatment



PARTITION

Divide into a grid the areas to be treated, using a white pencil and creating areas that are not too large

6





7

APPLICATION

Distribute on the interested area a quantity of iSol. Laser Gel conductor to help the sliding of the handpiece

PROTECT

Isolate the emitter area perfectly with transparent film in order not to directly touch the client's skin and guarantee perfect hygiene

8



9

SHOT

Make the spots on the interested areas paying attention to any reactions from the customer



Y S T E P

REMOVE

Remove all the used gel layer from the area (can't be reuse) and proceed with an accurate cleaning of the treated area

10



11

SOOTHE

Apply a quantity of iSol. Beauty Laser Repair Lotion onto the area dabbing and massaging uniformly to soothe and improve the skin during the post treatment



12

TAKE OFF

Remove a couple of hairs from the treated area to demonstrate to the client the effectiveness of the treatment as, the hairs, they will remove without any pain








PORTABLE HD Laser 808nm

Some customers often require personalized and dedicated service, with Portable Resolution Epilation HD Laser we can increase our services adding levels of excellence often difficult to reach. Practical, handy and without diminishing its effectiveness, the Portable Resolution Epilation HD Laser becomes an exceptional and unique business tool. No worries with this new equipment, not even in terms of protection, since it comes with a durable shell that protects it from accidental bumps. Unwanted hair has always been a big problem and you can not reach a beauty center? Now you will have no more excuses!

2195-8

800W

-  High Resolution Laser Matrix
-  Regular Focus Sapphire 13x13mm
-  Comfort Effect
-  12 spots per second
-  Memory Card
-  Technical Evaluation



Size in mm L490 x D250 x H690

Ref. Handpiece: 9931-8

- ✓ Be protagonist of your job!
- ✓ Expand your business!
- ✓ Always at your fingertips!
- ✓ Exclusive professionalism!



High Resolution Laser Matrix

All new generation laser equipment uses a wavelength of 808nm which corresponds to a laser beam of pure red light. Particularly effective on the pigments of the hair bulb that absorb the light up to about 98%. Thanks to the increased power 800W we will have a further precision in hitting the hair.



Regular Focus Sapphire

REGULAR: thanks to its 800W of power it allows to get a decisive action on the area to be treated. In addition, the greater reduction in pulse width combined with the Comfort Effect cooling system allows more comfortable treatments.



12 spots per second

With a view to optimization and improvement of the performance, the execution of the Overline Laser treatment, provides the use of a multispot methodology that allows up to 12 spots in a fraction of a second. All this puts the aesthetic professional operator in conditions to be able to work safely and improve time and effectiveness of the treatment and its final result.



Comfort Effect

It simply means that we have inserted a cooling system that makes the experience of Overline epilation, a pleasant sensation of light freshness and comfort for the skin



Memory Card

The memory card is nothing but a practical and effective "Personal Card" that allows you to store any type of information related to the type of treatment we are carrying out and possibly customize it or run it instantly. To be able to have access to data within the card, all you have to do is insert it into the slot



Technical Evaluation

With our 8" touch screen we can customize any treatment thanks to a detailed evaluation. The software offers the possibility for the professional operator to enter all the personal data of our customer related to the epilation service. We are able to recognize four important families: the phototype according to Fitzpatrick, the color of the hair, the size of the hair and finally the piliferous density



OVERLINE LASER COMPARED



Overline reserves the right to make changes to the equipment described, at any time and without notice. Overline equipment is designed and programmed to obtain results on imperfections without passing the allowed aesthetic limit.




new

TAKE CARE OF
YOUR WORK
DON'T WORRY
ABOUT THE
REST,
OVERLINE IS
THERE FOR
YOU

Overline equipment (2190-12L,
2190-12, 2190-10, 2190-08 and
2195-8) and the handpieces of the
same, are guaranteed by defects
in material and labor, ex works, for
a period of 36 months*.

*See General Conditions of Sale

The equipment respects:
TECHNICAL INFORMATION SHEET n.21b category: AESTHETIC LASER DEFOCALIZED FOR DEPILATION List of appliances: LASER FOR AESTHETIC DE-
PILATION Decree of October 15th 2015, n. 206. Regulation containing modifications to the Decree of May 12th 2011, n.110, concerning the regulation for the
implementation of article 10, paragraph 1, of the Law January 4th, 1990, n.1, relating to electromechanical equipment used for the beautician activity. 28-12-2015
OFFICIAL JOURNAL OF THE ITALIAN REPUBLIC General Series - n. 300 Entry into force on 12.01.2016

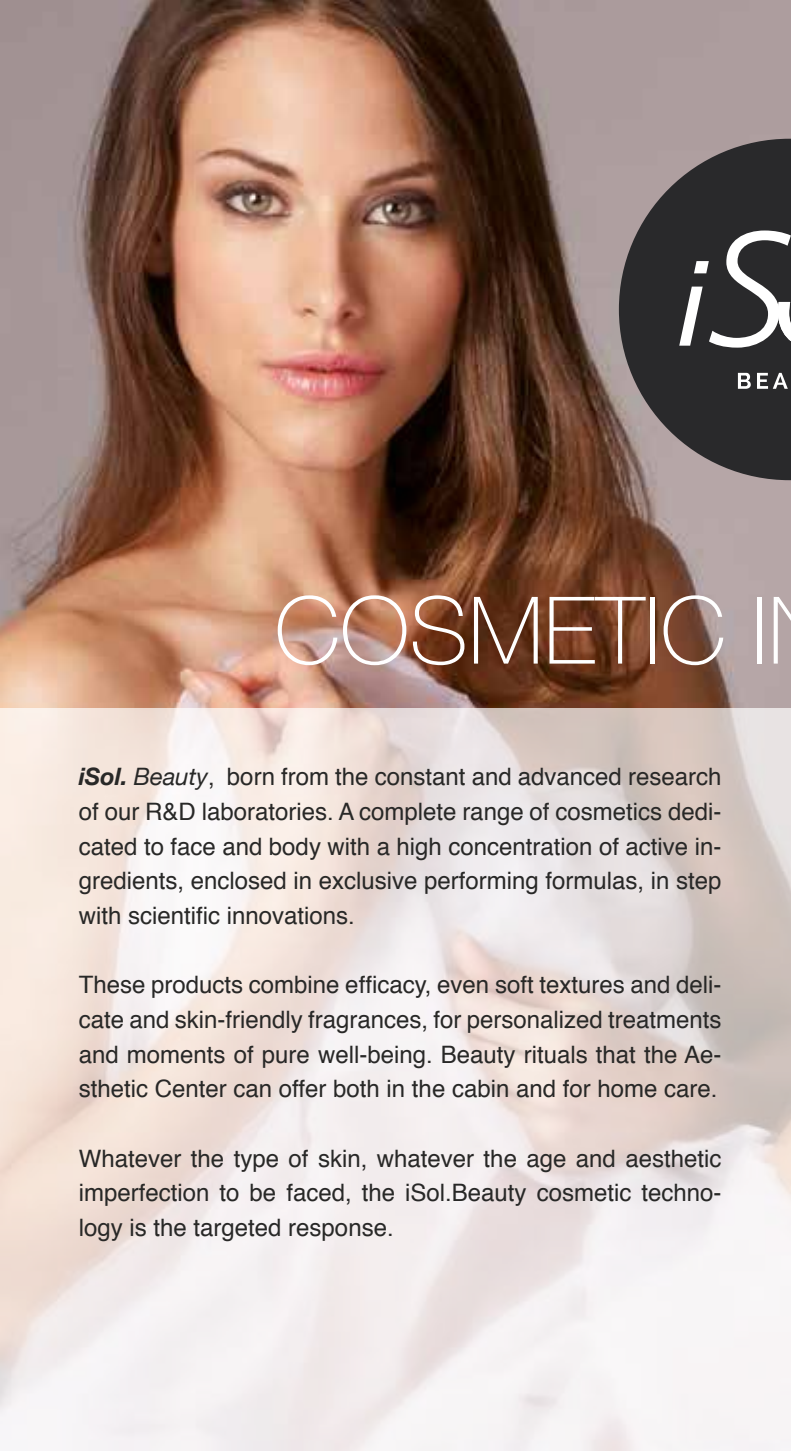

GOSMETIC INNOVATIONS

iSol. Beauty, born from the constant and advanced research of our R&D laboratories. A complete range of cosmetics dedicated to face and body with a high concentration of active ingredients, enclosed in exclusive performing formulas, in step with scientific innovations.

These products combine efficacy, even soft textures and delicate and skin-friendly fragrances, for personalized treatments and moments of pure well-being. Beauty rituals that the Aesthetic Center can offer both in the cabin and for home care.

Whatever the type of skin, whatever the age and aesthetic imperfection to be faced, the *iSol. Beauty* cosmetic technology is the targeted response.

iSol. Beauty is the newest collection of face, body and hair removal cosmetics, created to interact perfectly with the most modern aesthetic devices and increase the result. A line that, in addition to professional products, also provides special formats for home care. Innovative formulations that thanks to the Liquid Crystal Emulsifier, Red Grape Stem Cells, Tri-complex Hyaluronic Acid, Ozonized Argan Oil, Tri-Peptide and Micro lipids of Caffeine, Ivy and Algae Laminaria, are in able to prolong the bioavailability of the active ingredients, increase their capacity for absorption and diffusion on the epidermis, favoring the natural mechanisms of skin renewal, while simultaneously performing a nourishing, moisturizing, antioxidant and energizing action.

The dermatological tests conducted by the University of Ferrara guarantee maximum tolerability, excellent quality, high performances and total safety of cosmetics. The soft texture of the products and the delicate fragrances without allergens make the treatment pleasant and relaxing, for absolute comfort

reliable
effective
dedicated
to the skin

- Working methods in synergy with the technologies
- Better absorption of cosmetics
- More efficacy, more results
- Home care line
- Fragrances allergens free
- Quality and safety dermatologically tested
- Soft texture of cosmetics
- Maximum comfort of the skin



All iSol cosmetics. Special Beauty are dermatologically tested at the University of Ferrara which guarantees the quality and the high level of tolerability

The fragrances are selected and tested without allergens to respect the skin and ensure maximum tranquility in the use of the products.

Enhance the Benefits of Laser Treatments



LASER GEL

Specific lenitive conductive gel for LASER treatment with rose water distillate

ISO.LASER.100 - 4500ml / 152,16oz (Cabin)

Soft emollient conductive gel enriched with precious rose water distillate, with toning and soothing properties, specific for LASER treatments, which can also be used with other suitable equipment.



LASER REPAIR LOTION

Post LASER repairing emulsion

ISO.LASER.200 - 250ml / 8,45fl. oz (Cabin)

Emulsion containing Betaglucan which stimulates the natural skin defense processes, Allantoin with repairing action and Vitamin E with antioxidant action, enriched with other moisturizing, emollient and soothing active ingredients such as Aloe and Calendula Mucilage.



JOJOBA FACE & BODY SCRUB

Jojoba Microgranules Gel Scrub

ISO.SCRUB.200 - 250ml / 8,45fl.oz (Cabin)

Scrub with micro-granules of jojoba that soften during the application massage, thus modulating the action over time. It allows a high performance of skin exfoliation and a smoothing and protective action. Gentle and for all skin types.



CLEANSING SOFT MOUSSE

detergente Dermocompatibile Viso & Corpo

ISO.FOAM.150 - 150ml / 5,11fl.oz (Cabin & Home Care))

Innovative 2 in 1 cleanser (milk & tonic) in soft mousse, suitable for all skin types, to be used before each treatment or as a daily cleaning at home with moisturizing action and leaving the skin soft and silky.



always at
the service
of beautician

OVERLINE

Avant-garde and Innovation are the ideal terminologies to identify the Overline Company. A historic Milan reality, undisputed leader in the production and marketing of hi-tech equipment for professional aesthetics.

It was born in 1989 and since then it can be simply described "A success story" that expresses one's style business through an ethical commitment and a commercial management attentive and responsible.



OVERLINE
ITALIA

A 100% Made in Italy production, which offers competitive advantages in terms of performance, safety and service to the customer. Overline technologies are distributed in the best beauty centers and beauty institutes throughout the Italian territory and in all European countries as well as in South America, Mexico, Africa and Asia



The terminologies adopted in the company information materials, to define technologies and characteristics, are those allowed on an international level and commonly used.

RESEARCH & DEVELOPMENT

Heart of the company, the Innovative Laboratory Research & Development where high performance technologies and simple use, provide effective answers to the needs of professional aesthetics operators. They are different systems, safer and multiple features for both face and body but also for epilation.



reliability
research
safety



CERTIFICATIONS



Overline is a Technology S.r.l. Brand, a company that engages itself constantly in the research and development of technologies, to offer to the professional beauty market, innovative systems that give visible results but at the same time respect the maximum safety of the operator and the customer. Today, Technology, thanks to its great experience, has obtained important certifications attesting the fullness compliance with international regulations and standards in production of biomedical equipment, electro-aesthetics and cosmetics. The certifications that make all the technologies at the highest level are the following:

Tüv Rheinland ISO 9001:2015. Represents the certification of the Quality Management System, recognized at international level, ensures that the equipment meets the management system required by law for the design, the production, the sales, the development and the research.

Tüv Rheinland EN ISO 13485:2016. The certification related to the Quality Management System for Medical Devices, it is a standard of Quality reference in the regulatory procedures that govern design, production and the placing on the market of medical devices.

Tüv Rheinland OHSAS 18001:2007. Relative to the standard for the management of health and safety of workers.

OVERLINE

ITALIA

We are associated with FAPIB



Overline reserves the right to make changes to the equipment described, at any time and without notice. **Overline** equipment is designed and programmed to obtain results on imperfections without passing the allowed aesthetic limit.



For further information please contact us at number **+39 02 53 95 215** or visit our website **www.overline.it** write to **info@overline.it** and follow us on our Social Page

